

Rely on iScreen **OFD** for **dependable** saliva screening—**anywhere, anytime.**

FEATURES/BENEFITS

- *Eliminates need for controlled urine collection site*
- *Can be used anywhere, at any time, eliminating privacy concerns and same sex collectors*
- *Results in under 10 minutes*
- *Available in 4, 5, and 6 drug test panels*
- *Simple procedure*




AVAILABLE TEST PANELS

4 Panel — COC, THC, OPI, M-AMP (P/N: 2029)

5 Panel — COC, THC, OPI, M-AMP, AMP (P/N: 2024)

6 Panel — COC, THC, OPI, M-AMP, AMP, PCP (P/N: 2025)


iSCREEN OFD COMPONENTS

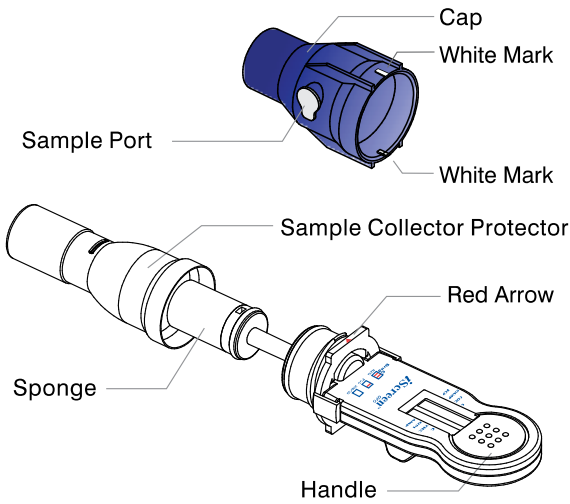

iSCREEN OFD TESTING PROCEDURE


1 Instruct the donor not to place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.

2 Remove Sample Collector Protector.

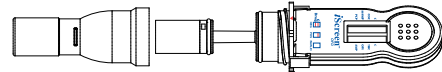

3 Insert Sponge end of collector into mouth.

Actively swab the inside of the mouth and tongue with Sponge. As soon as the Sponge softens slightly, gently press the Sponge between the tongue and teeth to ensure complete saturation. The Sponge is saturated when no hard spots can be felt. Collect for at least three (3) minutes before removing Sponge.

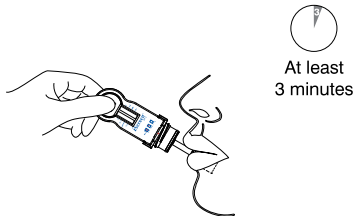


Hand the saturated collector over to the tester.


4 Align the Red Arrow on the device with either one of the White Marks on the Cap. Insert the collector vertically into the Cap and press down firmly until the Cap reaches the thread.

5 Twist the Handle clockwise 180° to tighten the Cap until the Red Arrow lines up with the other White Mark.

6 Place the test device horizontally on a clean and level surface with result window facing up.

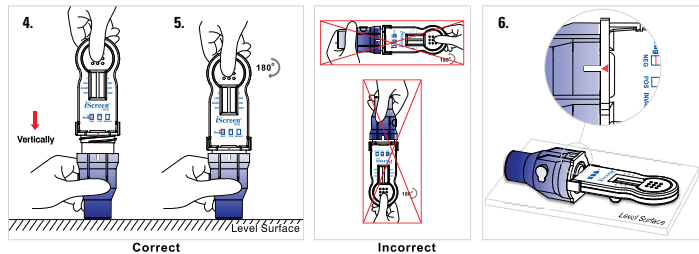

7 Read results at ten (10) minutes. Do not read results after 1 hour.

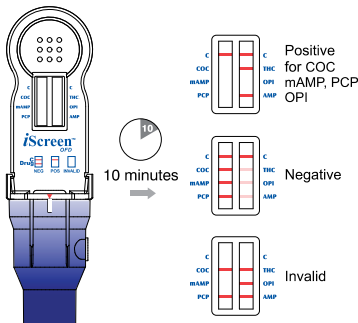

*NOTE: ANY POSITIVE RESULT OBTAINED WITH THIS SALIVA SCREENING TEST IS PRESUMPTIVE AND SHOULD BE CONFIRMED BY AN ALTERNATE METHOD SUCH AS GC/MS. CONTACT OUR LAB FOR ADDITIONAL GC/MS CONFIRMATION SERVICES.

For forensic use only.

*Marijuana (THC), Cocaine, Opiates (300), Phencyclidine, Methamphetamines, Amphetamines

Call today: **877-444-0049**