

## 6 Cassette / substance abuse screening device



### Drugs tested:


- *Amphetamine*
- *Cocaine*
- *Marijuana*
- *Methamphetamine*
- *Opiates*
- *Phencyclidine*

Reliable and convenient **CLIA**  
waived on-site drug screening.

reditest®

PRODUCT OVERVIEW

## 6 CASSETTE COMPONENTS

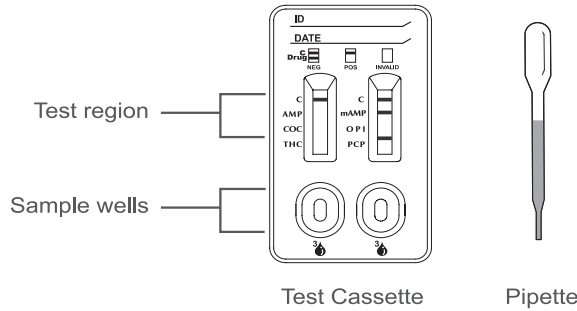


## TESTING PROCEDURE

Allow the test device, urine sample, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

- 1 Bring the pouch to room temperature before opening it. Remove the test device from the sealed pouch. Use test device as soon as possible.

- 2 Place the test device on a clean and level surface. Hold the dropper vertically and transfer 3 full drops of urine (approx. 100 µL total volume) to each of the sample well (S) of the test device.

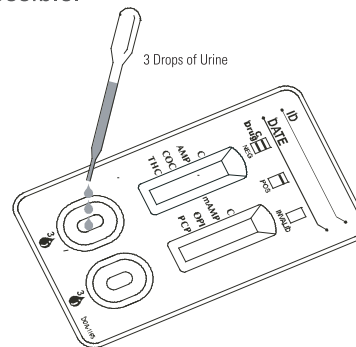

Start the timer. Avoid trapping air bubbles in the sample well (S). See illustration.

- 3 Wait for the colored line(s) to appear. The results should be read at 5 minutes or up to 4 hours after test initiation.

### RESULT KEY

C				
Drug				
	NEG	POS	INVALID	INVALID

\*NOTE: ANY POSITIVE RESULT OBTAINED WITH THIS URINE SCREENING TEST IS PRESUMPTIVE AND SHOULD BE CONFIRMED BY AN ALTERNATE METHOD SUCH AS GC/MS. CONTACT OUR LAB FOR ADDITIONAL GC/MS CONFIRMATION SERVICES.

For professional in vitro diagnostic use only.

Call today: **877-444-0049**