


# Panel-Dip / substance abuse screening device

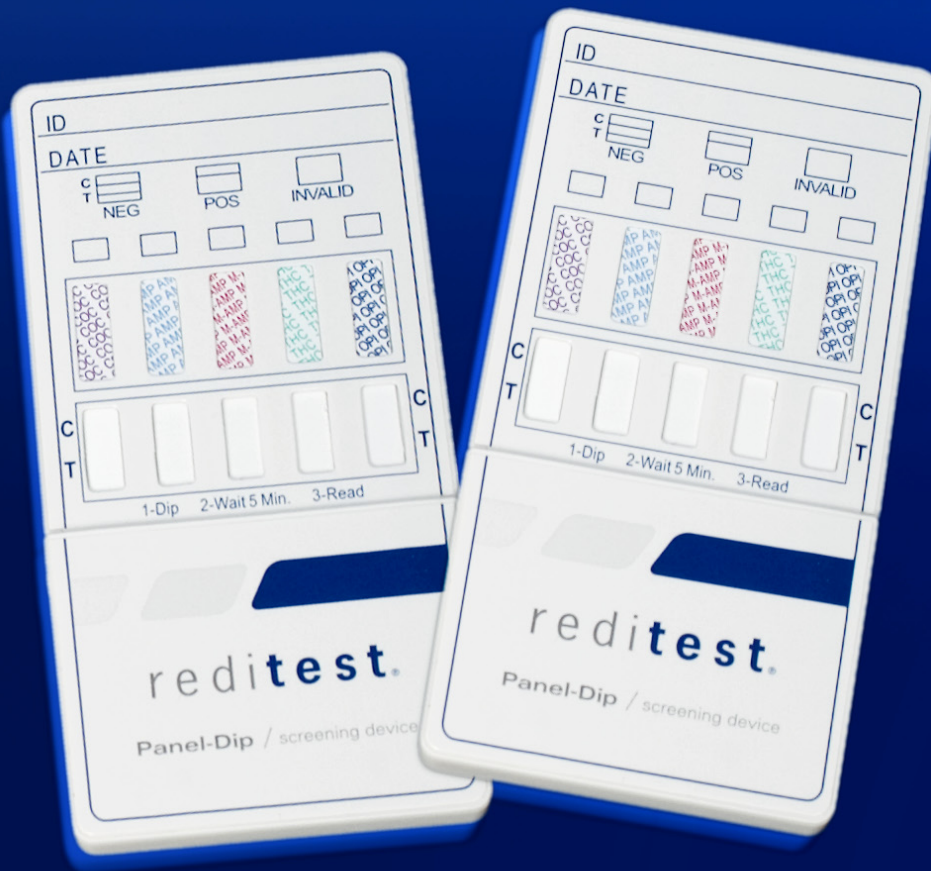

**A reliable and cost effective  
on-site drug screening solution.**


## Available tests include:

- *Amphetamine*
- *Methamphetamine*
- *Barbiturates*
- *Benzodiazepines*
- *Buprenorphine*
- *Cocaine*
- *Marijuana (THC)*
- *MDMA (Ecstasy)*
- *Methadone*
- *Opiates*
- *Oxycodone*
- *Phencyclidine (PCP)*
- *Tricyclic Anti-depressants*

**reditest®**


PRODUCT OVERVIEW

## PANEL-DIP COMPONENTS

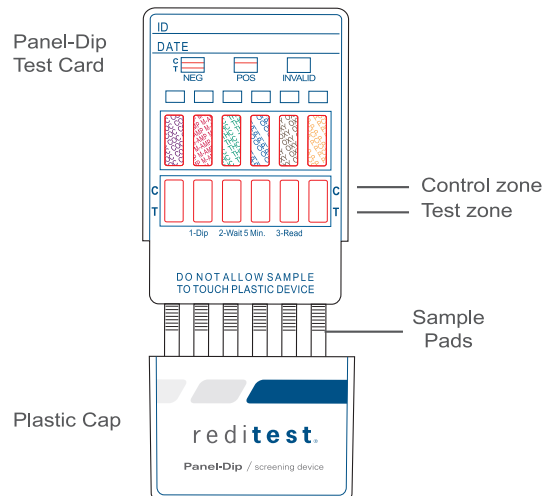

## TESTING PROCEDURE

- 1 Remove device from foil pouch.
- 2 Pull plastic cap from test device.
- 3 Dip revealed sample pads into urine sample for a minimum of 15-30 seconds until urine is seen wicking into the reading zone.
- 4 Remove the device from the urine and replace the plastic cap. Place on a flat surface.
- 5 Read test results at 5 minutes. (Do not interpret results after 4 hours as false results may occur. Note: Any line at all in the test zone is to be interpreted as negative.)

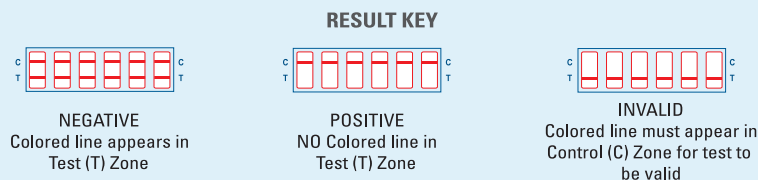

\*NOTE: ANY POSITIVE RESULT OBTAINED WITH THIS URINE SCREENING TEST IS PRESUMPTIVE AND SHOULD BE CONFIRMED BY AN ALTERNATE METHOD SUCH AS GC/MS. CONTACT OUR LAB FOR ADDITIONAL GC/MS CONFIRMATION SERVICES.

For professional in vitro diagnostic use only.

Call today: **877-444-0049**