

# RediTest Cassette

## Product Procedure

For Professional In Vitro Diagnostic Use Only

See product insert for full details on instructions, limitations, and warnings.


## RediTest Cassette | Drugs of Abuse Test Device

### INSTRUCTIONS FOR USE

Allow the test device, urine sample, and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

**1** Bring the pouch to room temperature before opening it. Remove the test device from the sealed pouch. Use test device as soon as possible.


**2** Place the test device on a clean and level surface. Hold the dropper vertically and transfer 3 full drops of urine (approx. 100  $\mu$ L total volume) to the sample well (S) of the test device. Start the timer. Avoid trapping air bubbles in the sample well (S). See illustration.

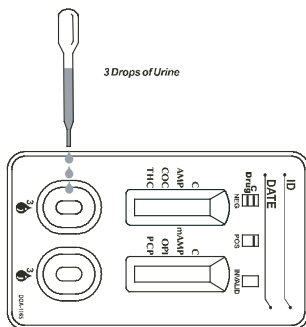

**3** Wait for the colored line(s) to appear. The results should be read at 5 minutes or up to 4 hours after test initiation.

See reverse side for Result Interpretation.

## RESULT INTERPRETATION

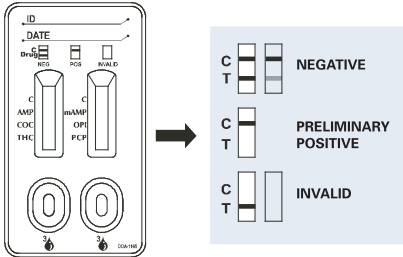

### POSITIVE:

A colored line appears in the control region (C).

*NO line appears in the test region next to the name of a specific drug tested. The positive result means that the drug concentration in the urine sample is greater than the designated cut-off for a specific drug.*

### NEGATIVE:

A colored line appears in the control region (C) and a colored line appears in the test region (T).

*Up to four colored lines may appear in each result window. One line will be in the control region (C). Up to three lines will be next to the drug names in the test region. This negative result means that the drug concentration in the urine sample is below the designated cut-off for a specific drug. \*NOTE: The shade of color in the test region (T) may vary. The result should be considered negative whenever there is even a faint colored line.*

### INVALID:

No line appears in the control region (C).

*If this occurs read the directions again and repeat the test with a new panel. If the result is still invalid stop using the test kit and contact your distributor.*

**NOTE: ALL POSITIVE RESULTS ARE PRESUMPTIVE AND SHOULD BE CONFIRMED BY AN ALTERNATIVE METHOD (e.g. GC/MS)**